


ADVANCED BLOOD PRESSURE MONITOR WITH ECG

TS 320BPEC




Connect TS 320BPEC via Bluetooth on Smartphones and Tablets

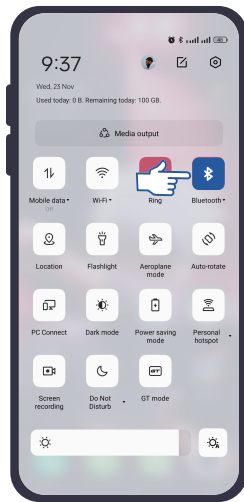


1. Turn on Mobile Bluetooth




2. Open iBP ECG Plus App

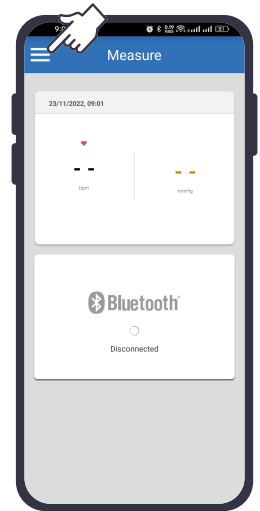


3. Touch the top three lines to open more options

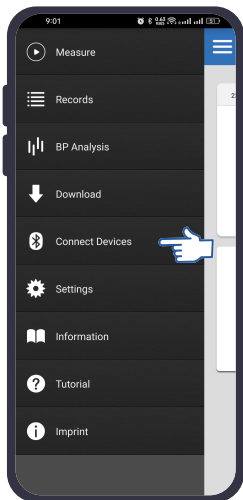


4. Go to Connect Devices

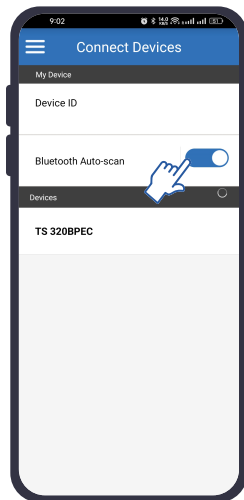

5. Turn on Auto-scan & Tap TS 320BPEC to establish connection.

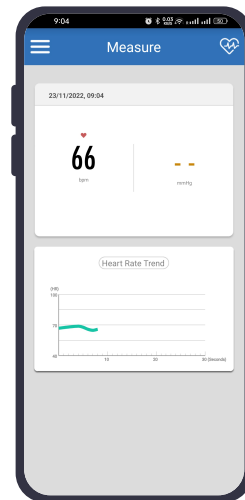

6. Start your measurement
Turn page to measurement Procedure

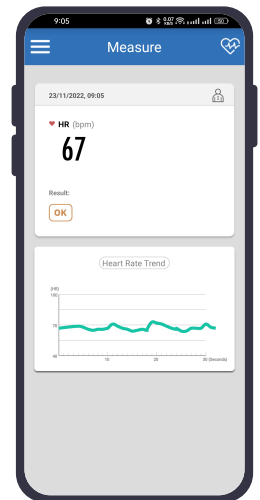


7. Result appear approximately 30 seconds.

Bluetooth 4.0 technology


With Bluetooth 4.0 technology, this device ideally provides real-time display of your measurement on Smartphones/Tablets for a great portability of reviewing your records anytime, anywhere.

Taking TS 320BPEC Measurement Procedure


- ✓ Connect the cable connector of ECG Pen to the port on the right side of the device.

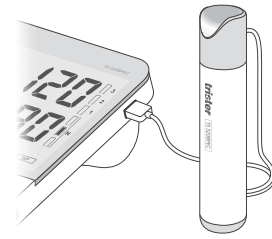


- ✓ Lay your right index finger on the hollow part of top electrode and hold the ECG Pen in your right hand.

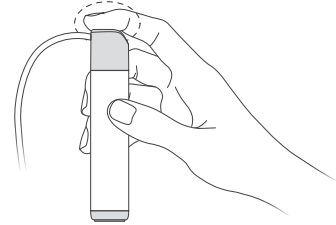


- ✓ Place the bottom electrodes on bare skin about 5 centimeters (2 inches) below your left nipple supported by the ribcage. Press the ECG Pen GENTLY until you hear the click sound.

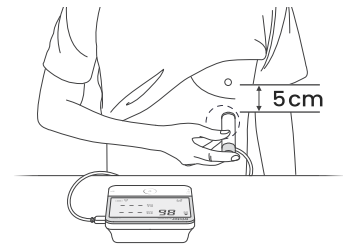

- ✓ **Note :** The force should be kept constant during the measuring period. Do not press the electrodes hard. Excessive muscle tension might affect the accuracy of measurement result.

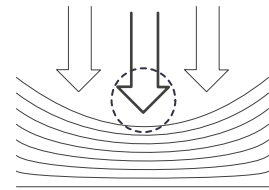

- ✓ The Digits Display shows 30 seconds of count down and the **ECG** symbol will stay on after the ECG measurement start.

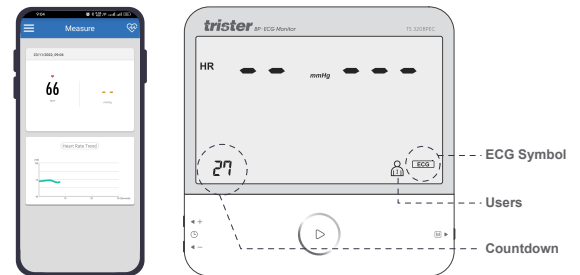

- ✓ During ECG measurement :

- Measuring period will count down from 30 seconds.
- The heart rate digits will dynamically show the real-time heartbeat.

- ✓ **Note:** The Averaged Heart Rate will only be displayed once the measurement is completed.